

## **HXJH atomizer set**

Packing per box: 1 piece of atomizer and 2 pcs of cartridges

Validity period: 12 months

Production date: See packaging Year/Month/Date

### **Atomizer Parameters:**

Charging interface: TYPE C

Charging voltage: 5V (Do not use a charger with a voltage higher than 5V for charging)

Charging current: 330mAh

Battery capacity: 420mAh

Charging time: about 45 minutes

Number of smoking times available when it is fully charged:

350-400 smoking times

Net weight: 22.5g

Packing box contain: Atomizer rod (420mAh) x 1 / cartridges x 2



### **Instructions for use of HXJH Medical Atomizer:**

1. Open the package, remove the upper and lower silicone plugs and air intake stickers at the cartridge and then insert the cartridge into the atomizer. User will see there is a flash of light at the atomizer, then user can start to smoke at the opening part of the cartridge. When there is red light flashing at the atomizer, then this means it is out of battery and need to re-charge the atomizer.
2. Use TYPE C to recharge the atomizer. Then user will see the light flashing at the atomizer. About 45 minutes when the light is no longer flashing, then it means the atomizer if fully charged.
3. While using this atomizer, user inhale deeply into their lungs, hold their breath for 5 seconds, and then exhale through their nostrils;
4. Take 10-15 puffs continuously each time;
5. Each time after used, separate the cartridge from the atomizer. Put back on the silicone plug at the opening of the atomizer and cartridge.
6. There will be no damage or loss when the cartridge and atomizer are connected together and not in use.

## **HXJH Cartridge set**

Packing per box: 3 pcs of cartridges

Validity period: 3 years before unpacking and 3 months after unpacked

Production date: See packaging Year/Month/Date

### **Safety advice:**

1. Keep away from children and pets.
2. Store in a cool and dry place, away from direct sunlight.
3. This product is only suitable for oral atomization. Those who have allergic symptoms, pregnant and lactating women are contraindicated to use this product.

